
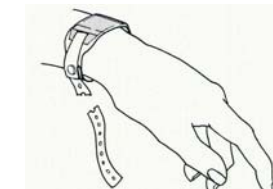
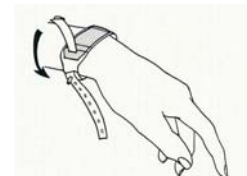
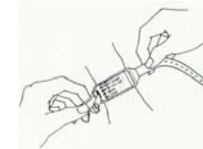
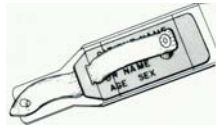
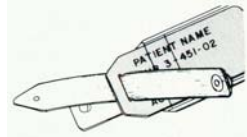


Bio-Logics Patient Identification Band Assembly Instructions



Assembling Band

1. Remove complete band unit (band, tether and throw away) by peeling from female clip end as shown.
2. Separate tether and throw away from band.
3. Insert tag ¾ way into ID band pouch. Be sure that the tag reads to you and that the slot in the tag is on top, exposed and near male clip as shown.
4. With female clip on underside of tether, insert pointed end through tag slot from front until tether halfway mark is located in tag slot.
5. Push male clip through hole located behind point on tether and slide tag deeper into pouch on ID band.
6. Lay ID band across wrist of patient. Orient information so that it reads to you.
7. Bring adjustment strap around wrist and position index finger under or between the male clip and patient wrist. Match up male clip with the corresponding adjustment hole and insert stem of clip through hole.
8. Fold adjustment strap back and push male clip stem through adjacent hole on strap as shown. With the adjustment strap secured in two positions, fold down the tether with female clip and snap clip over the stem on the male clip.
9. Tug on the excess strap or "tail" to be certain the clip is securely fastened. Remove excess adjustment strap by cutting strap just below clip.

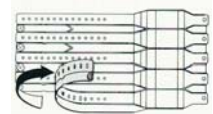


Adding Allergy Alert

1. Remove a complete Allergy Alert Tag from roll by breaking at perforations. A complete tag is approximately 6 ½" long.
2. With printing right side up, legibly write patient's allergies and/ or drug sensitivities on both halves of tag using a water-proof ink.
3. Peel off the larger of the two section (left side) marked with the larger printing of "Allergy" plus the word "Chart" and stick to the outside surface of the patient's chart cover.

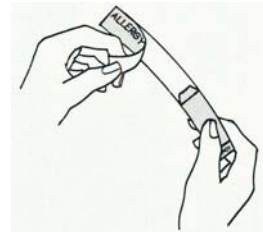
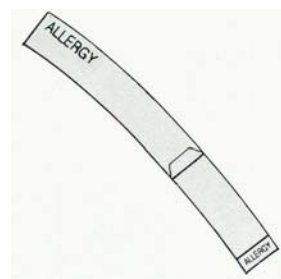


NOTE: DO NOT tear the liner or backing paper when removing this label, as it will be needed to help insert the other portion into the ID band.

4. Fold the liner back, away from the remaining label at the crease behind the black line at the base of the point on the tag. This exposes approximately ¼' of adhesive on the backside of the point. Firmly crease the fold.

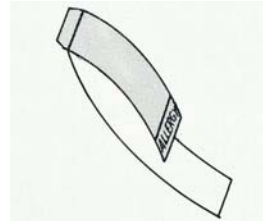

5. Locate the allergy pouch on the ID band. It is directly underneath the pouch that carries the plastic ID tag, but opens at the other end where the band tapers to the adjustment strap width. (Middle opening)
6. Remove the ID tag from its pouch and let dangle. Insert index finger between wrist and ID band underneath opening of allergy pouch. Pull so that pouch is relatively flat, not curved.
7. Insert the folded end of the allergy tag into pouch with sticky point leading. Insert tag until it is completely covered by vinyl and the line above the word allergy at the end lines up with the end of the ID tag pouch above.
8. Pinch the ¼" exposed adhesive with thumb and index finger, sandwiching it in the vinyl. With the other hand pull on tag liner and peel it away from the tag in the pouch.
9. Replace ID tag in its own pouch on top.

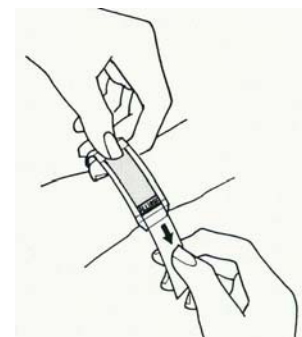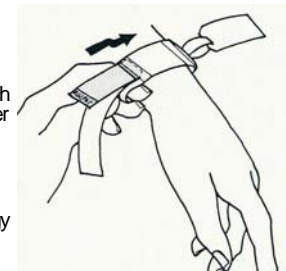

Note: Tag with "ALLERGY" should extend beyond bottom of plastic ID tag.

Removing/Reattaching Band

- To remove band, cut band off.
- Keep luggage tag portion of tag. (The hard plastic part with the patient identification)
- To reattach, assemble new band using existing luggage tag.
- It is very important the existing luggage tag is reused if the band is removed to prevent errors with positive identification
- Reattach new band as soon as possible

Adding Allergy Alert Tips:

- The allergy pouch opens only at one side of the wristband.
- It opens on the opposite end from where the ID tag slides in.
- It opens on the opposite end of the snap.
- It opens on the side where the adjustment holes are located.
- It goes in the middle opening (The first is for the ID tag, the 2nd is for the allergy alert, the 3rd is for the DNR alert)