



Conf-ID-ent™ Blood Band

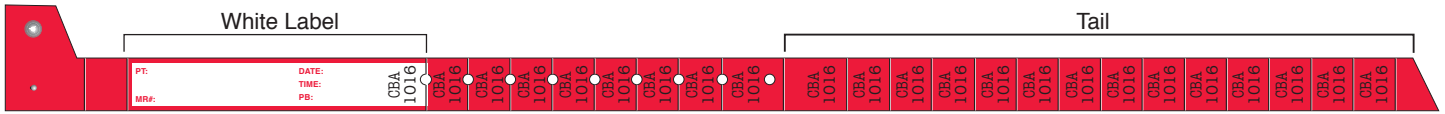

Directions for use


STEP 1

Using a ball point pen, complete information on white label.

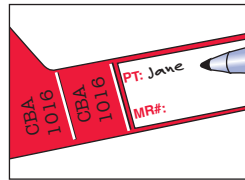

STEP 2

Remove tail of wristband at the perforation.

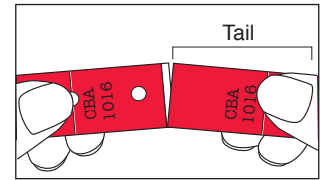

STEP 3

A) Apply white patient identification label to the draw tube.




B) Using the adhesive on the back of the tail, secure to the draw tube.




STEP 4

Adjust wristband securely around patient's wrist.

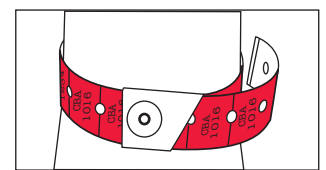

STEP 5

If needed, utilize white band for additional length.

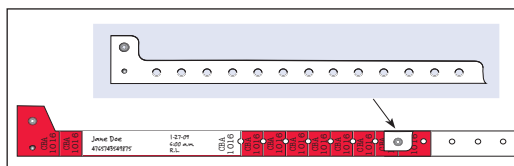

STEP 6

Draw blood into labeled tube and transport to laboratory.



STEP 7

Before transfusions, compare serial number on blood label with number on band. Always check two locations on wristband as carbon copy numbers can become difficult to read.

For a complete selection of Conf-ID-ent™ Blood Bands, visit us online at www.patientIDexpert.com