


**STEP 1**

Print ID band on laser printer

**STEP 2**

Push down here to begin separating imaged band from sheet.



**STEP 3**

Turn sheet over and lift ID band from sheet – start from the right and pull to the left.



**STEP 4**

Fold film over imaged area to create ID band.



SELF LAMINATING WRIST BAND



**WRIST**

SELF LAMINATING WRIST BAND



**ANKLE**

THE *St. John*™ COMPANIES  
 Reorder: Call (800) 435-4242  
 Cat. No. WBW21