

STEP 1

Print ID band on laser printer

SELF LAMINATING WRIST BAND

STEP 2

Push down here to begin separating imaged band from sheet.



WRIST

STEP 3

Turn sheet over and lift ID band from sheet – start from the right and pull to the left.



THE *S. John* COMPANIES
Conf-ID=ent
 Patient Identification System
 Reorder: Call (800) 435-4242
 Cat. No. WBW22

STEP 4

Fold film over imaged area to create ID band.



ANKLE

SELF LAMINATING WRIST BAND

Licensed under Laser Band U.S. Patent #6,438,861, #6,067,739, #6,000,160, #5,933,993 and others pending.

Back side is Tamper Evident

